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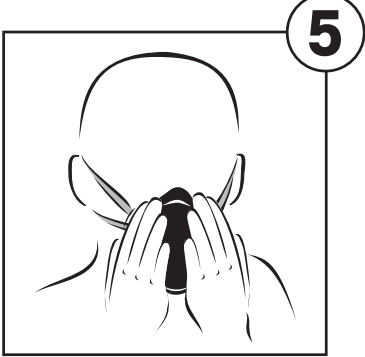
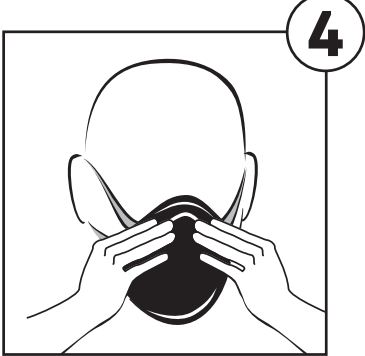
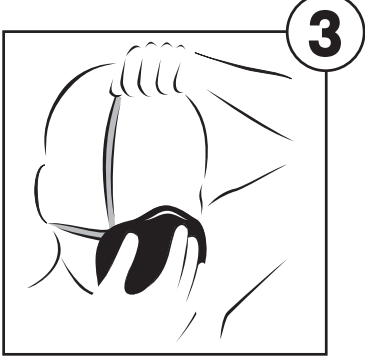
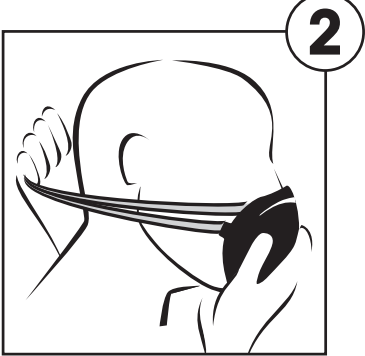
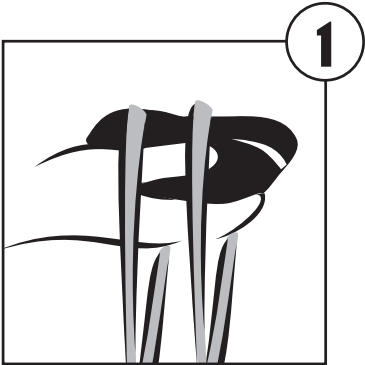
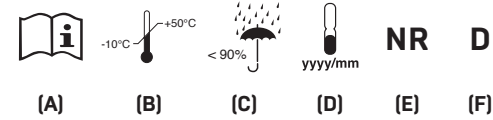
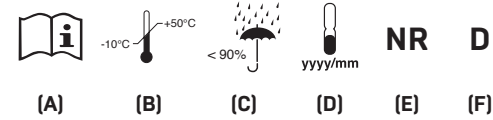
FILTERING HALF MASKS AGAINST PARTICLES

Manufactured/Imported by: Refer to packaging for contact details.

CE certification according to the (EU) Regulation 2016/425 and subsequent amendments. Certified by BSI Group The Netherlands B.V. Say Building, John M. Keynesplein 9, 1066 EP Amsterdam, Netherlands, n° 2797.

UKCA certification according to the Regulation 2016/425 and subsequent amendments. Certified by BSI Assurance UK Ltd, Kitemark Court, Davy Avenue, Knowlhill, Milton Keynes, MK5 8PP, United Kingdom, n° 0086.

The EU/UKCA conformity declaration is available at following address: www.nerispa.com

INFORMATIVE NOTE

IMPORTANT:
EN 149:2001 + A1:2009 Single-use respirators against solid and liquid aerosols are classified according to one of three categories: FFPI, FFP2, and FFP3.

Before reading the following information, please check the category to which the respirator belongs - this is indicated on the packaging and on the respirator.

APPLICATIONS:

FFPI: protection against non-toxic solid and liquid aerosols [e.g. oil-mists] in concentrations up to 4 x TLV, or 4 x WEL [APF=4].
FFP2: protection against non-toxic and low-to-coverage toxicity solid and liquid aerosols [e.g. oil-mists] in concentrations up to 12 x TLV, or 10 x WEL [APF=10].
FFP3: protection against non-toxic, low-to-coverage toxicity and high toxicity solid and liquid aerosols [e.g. oil-mists] in concentrations up to 50 x TLV, or 30 x WEL [APF=30].

CHECK PRIOR TO USE: Inspect respirator, including any component parts, for holes, tears or damage prior to use. Should any damage be identified, **DO NOT USE IT**. Get a new one.

FITTING INSTRUCTIONS:

1. Hold the respirator in hand with the nosepiece at your fingertips.
1. Allow headbands to hang freely below hand.
2. Take the mask in your hand and press it to your face with light pressure.
3. Take the rubber strap with the other hand and pull the lower strap over your head till under your ear. At the same time pull the upper strap to the upside of the head.
- Respirators with adjustable headbands only:** while maintaining the position of the respirator, adjust tension by pulling tab of each headband. [Tension may be decreased by pushing out the back of the buckle].
4. Finally fit the nose clip on your nose, so the mask fits as tightly as possible.
- 5a. To test fit for respirator without exhalation valve: cup both hands over the respirator and exhale vigorously. If air flows around your nose, reposition the respirator. If air leaks around the edges, repeat point 3 and 4.
- 5b. To test fit for respirator with exhalation valve: cup both hands over the respirator and inhale sharply. A negative pressure should be felt inside respirator. If air flows around your nose, reposition the respirator. If air leaks around the edges, repeat point 3 and 4.
- NOTE:** Change respirator immediately if breathing becomes difficult or respirator becomes damaged or distorted, or a proper face fit can not be maintained. Careful observance of these instructions is an important step in safe respirator use.

WARNINGS:

- A properly selected respirator is essential to protect user's health. Before selection, consult a trained person in prevention and protection against risks at workplace, for establishing respirator use suitability.
- Each user have to be trained to correct respirator employment before starting to use it.
- The non-observance of all instructions and limitations on the use of this respirator, for the whole duration of the exposure to contaminants, could reduce its effectiveness and cause diseases or permanent damages to the user.
- Neither the manufacturer nor the distributor can be held liable for an incorrect use of the respirator.
- Put on the mask and check its tightness before entering the contaminated area.
- Never alter or modify the mask in any way.
- This mask does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen level. Do not use in areas where oxygen concentration is less than 17 %.
- Do not use this mask to enter or stay in an area where contaminants or their concentrations are unknown or immediately dangerous to life or health.
- Do not use in explosive atmospheres or for fire-tighting operations.
- Beard, long sideboards and some respiratory characteristics may reduce the mask effectiveness and tightness.
- Discard and replace immediately the mask if it has been removed while in a contaminated area, if it is damaged and if breathing becomes difficult.
- Leave directly the contaminated area if breathing becomes difficult or in case of anxiety sense, vertigo and other distress.
- These filtering half masks are only for single shift use so they must be disposed at the end of the use. For this reason they don't need maintenance.
- Keep unused masks in their original closed boxes and store them in a dry noncontaminated area, away from direct sunlight.

MEANING OF PICTOGRAMS/SYMBOLS:

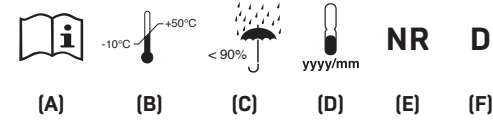
- [A]** See user information.
- [B]** Temperature range of storage condition.
- [C]** Maximum relative humidity of storage condition.
- [D]** End of shelf life. yyyy = Year; mm = Month
- [E]** NR: this respirator is limited to single shift use only (with a maximum of 8 hours) and shall not be re-used.
- [F]** D: the respirator passes the optional Dolomite clogging test for more comfortable breathing

NOTE:

TLV = Threshold Limit Value
WEL = Workplace Exposure Limit
APF = Assigned Protection Factor

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