NOTA INFORMATIVA cod. 390040 - mod. Latex Pro

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INFORMATIVE NOTE cod. 390040 - mod. Latex Pro

Please read this briefing note through carefully prior to use and prior to any maintenance work. The purpose of the information contained herein is to assist and direct the user in choosing and using the PPE.

Neither the manufacturer nor the distributor can be held liable for an incorrect use of the PPE. This briefing note must be kept for the duration of the PPE.

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The CE 0598 mark means that these gloves are personal protective equipment conforming to the essential health and safety requirements set forth in the (EU) Regulation 2016/425 and have been certified by the notified body: **SGS Fimko Oy P.O. Box 30 [Särkiniementie 3] 00211 Helsinki Finland, n° 0598.** In addition, the manufacturer is subject to the control CE of the finished product by the notified body **SGS Fimko Oy P.O. Box 30 [Särkiniementie 3] 00211 Helsinki Finland, n° 0598.**

OUR SIZE REFERENCES	SIZES CONTEMPLATED EN 420:2003+AI:2009
S	6.5
M	7
L	8
XL	8.5

The dexterity level of the fingers is equivalent to 5 out of 5.

The gloves meet the safety characteristics only if the size is right and worn properly.

RANGE OF USE

The glove is designed to protect the user against contact with chemical products and micro-organisms (intended as fungus, bacteria and viruses).

PICTOGRAMS AND PROTECTION LEVELS

 EN ISO 374-1:2016 / Type B	 EN ISO 374-5:2016
KLP	VIRUS
PROTECTION AGAINST CHEMICALS	PROTECTION AGAINST MICRO-ORGANISMS Resistance to air and water penetration using the method referred to in EN 374-2:2014 standards

Chemical	Permeation index (0-6)	Breakthrough time (min)	Degradation index
K Sodium hydroxide 40%	6	>480	20.7%
L Sulphuric acid 96%	3	>60	70.5%
O Ammonium hydroxide 25%	0	8	13.3%
P Hydrogen peroxide 30%	6	>480	-20.6%
T Formaldehyde 37%	1	>20	17.9%

Breakthrough time.

The time a chemical takes to pass through the glove material.

The values may differ when the chemical substance is used in a compound.

Do not use the gloves for handling the above-mentioned chemical products for longer than the times stated above.

	COMPLIANT WITH: REGULATION (EU) N. 10/2011 - REGULATION (EC) N. 1935/2004 - DIRECTIVES 85/572/EEC AND SUBSEQUENT AMENDMENTS.		
Ethanol 10%	Acetic acid 3%	Ethanol 95%	Polyphenyl oxide (MPP0)
Yes	Yes	Yes	No
10 min / 40°C	10 min / 40°C	10 min / 40°C	-

Since the performance levels are based on lab test results, they may differ from the actual conditions at the workplace. Consequently it is advisable to conduct a preliminary test of the gloves to see if they are suitable for their intended use.

The protective characteristics refer to new PPE, in good condition and that has never been subject to cleaning treatments.

USE

Prior to use, make sure the gloves are in good condition by which we mean no cuts, holes or cracks, etc. If the gloves are not in perfect condition, replace them immediatly.

The gloves must be used solely for the risks contemplated in this briefing note.

STORING

The gloves are packed in cardboard boxes.

Keep the gloves in their original packaging in a clean, dry place, away from sources of heat and direct sunlight. If stored as indicated, the gloves will keep their characteristics for a long time. The user must always inspect the gloves to ensure they are sound prior to using them.

Since these are disposable gloves to be used just the once, they must be replaced at the end of their use – they cannot be used a second time.

NOTES

Used gloves can be contaminated with chemicals or biological substances: dispose of them in compliance with the local regulations in force on this subject (dump, incinerator). People who are sensitive to rubber or latex should not use these gloves. If an allergic reaction occurs, stop using the gloves and seek medical advice. In the case of manufacturing defects this PPE will be replaced.

*** In the event of discrepancies between the different translations, the Italian version will esclusively apply.**

For more information, contact:

Registered Community Trade Mark, no 016928426 c/o EUIPO - Alicante - Spain

The EU conformity declaration is available at following address: **www.nerispa.com**

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